AXCENT ACRYLIC™

Dynasty Hi-Impact Pourable Denture Base

CADMIUM FREE

Processing Instructions

Complies with ISO 20795-1:2008 Specification Type 2, Class 2
Use with Garreco Dynasty Acrylics Self Cure Acrylic Liquid Monomer

INTENDED USE:

For the fabrication of dental appliances.

CAUTION:

This product is to be used only under the supervision of a licensed dental practitioner.

INDICATION(S) FOR USE:

Axcent Pour Type Denture Base is a Self-cure (with pressure vessel), pour-type acrylic resin for denture bases.

WARNINGS AND PRECAUTIONS:

IF ON SKIN: May be irritating to skin in some sensitive individuals, especially after prolonged and/or repeated contact. IF INHALED: May be irritated by gross overexposure, no matter how generated. IF SWALLOWED: Not expected to cause any harmful effects. Ingestion of large amounts may be irritating if product is swallowed, may cause nausea, headache, vomiting and/or diarrhea. IF IN EYES: May be irritating to the eyes by gross overexposure, no matter how generated. Symptoms of overexposure may include redness, itching, irritation and watering. Keep dust out of eyes.

CONTRAINDICATION(S):

Do not use with patients with a known allergy to methacrylates or other components of the product.

ADVERSE REACTION(S):

Reversible inflammatory changes of the oral mucosa may occur with susceptible persons.

INSTRUCTIONS:

1. Shake powder container to redistribute fibers, prior to measuring.
2. Carefully measure by volume 2 parts pour type powder to 1 part pour type liquid (40 g powder / 28 ml liquid). Add powder to liquid.
3. Spatulate for 20 seconds or until thoroughly mixed. Over spatulation will produce unwanted air pockets in the mix.
4. Immediately pour Axcent Pour Type Acrylic into sprue hole. Pour in a continuous stream to avoid entrapping air and continue pouring until the vent hole is filled.

5. Place flask in curing unit (with sprues upright). Add enough (130°F-54°C) water to submerge the flask to 3/4 of its height.

6. Cure for 20-30 minutes at 30 psi.

7. Remove flask from curing unit, de-flask the appliance and finish according to standard finishing techniques.

Repairs

Repairs to processed Pour Type Denture Base material can be made with Denture Base Repair Material, following the instructions provided. Alternatively, the repair may be processed using the original Pour Type Denture Base material from which the denture was processed. Pack and cure the repair according to Pour Type Denture Base techniques.

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Garreco, LLC