



ULTRA-SONIC CLEANER #4 TEMPORARY CEMENT REMOVER

Pemaco brand Ultra-Sonic Cleaner #4 is especially formulated to remove temporary cement from dentures, bridges, crowns, orthodontic appliances, and other dental restorations.

- Removes temporary cement on dental appliances
- Fast working

Indication for use: Removes temporary cement from dental appliances

Standard Sizes Available: 1 Gallon (3.8 Liters)

Directions: Place item in covered glass beaker of undiluted Ultra-Sonic Cleaner #4 solution. Immerse covered glass beaker in Ultra-Sonic Cleaner #1 solution. Ultra-Sonic Cleaner #1 acts as a conductor between tank and beaker. **DO NOT USE ON ALUMINUM.** Cleans in 5-10 minutes. Rinse items with water after cleaning. Keep beaker covered. Use in glass beaker only.

Item #: 3630040PM

Manufactured by Garreco, LLC
430 Hiram Road, Heber Springs, AR 72543, USA
Toll Free: (800) 334-1443
garreco.com



RELATED PRODUCTS

Ultra-Sonic Cleaner #1: General Purpose Concentrate - 3570040PM

Ultra-Sonic Cleaner #2: Tartar and Stain Remover - 3590040PM

Ultra-Sonic Cleaner #5: Plaster and Stone Remover - 3610040PM